

Quick Reference Collection Chart

Vancouver Island

>>>		Each of the following tests must be collected in a separate 3.5mL SST gold top tubes		
		Free Calcium (Ionized Calcium)		
		Phenytoin (Dilantin)		
		Mix 5-6 times		
>>>		Each of the following tests must be collected in a separate 5.0mL SST gold top tubes		
		Anti-Mullerian Hormone		Specific Allergen IgE (SAIGE)
		βHCG		
		Carotene		
		Mix 5-6 times		
		The following tests can be combined in one 5.0mL SST gold top tube		
		Cortisol	Free T4	Prolactin
		Estradiol	FSH	TSH
		Ferritin	LH	Vit B12
		Free T3	Progesterone	
>>>		The following tests can be combined in one 5.0mL SST gold top tube		
		Alpha Fetoprotein	C-Telopeptide	Thyroid Peroxidase
		Anti Thyroglobulin Ab	DHEAS	
		CA 125	Growth Hormone	
		CA 153	Insulin	
		CA 199	Thyroglobulin	
		Mix 5-6 times		
		The following tests can be combined in one 4.0mL EDTA lavender top tube		
		ESR		
		Hematology Profile		
		Malaria/Blood Parasites		
		Hemoglobin A1C		
		Mix 8-10 times		
		Each of the following tests must be collected in one 6.0mL EDTA lavender top tube		
		Hep B DNA		
		Hep C Viral Load		
		HIV Viral Load		
		Mix 8-10 times		

This document does not include requirements for all tests. For tests not listed, and complete processing and handling instructions, consult the online Laboratory Test Information (LTI) at www.lifelabs.com